



Primary Tip:

F11



*Key toggles browser
full screen on/off*

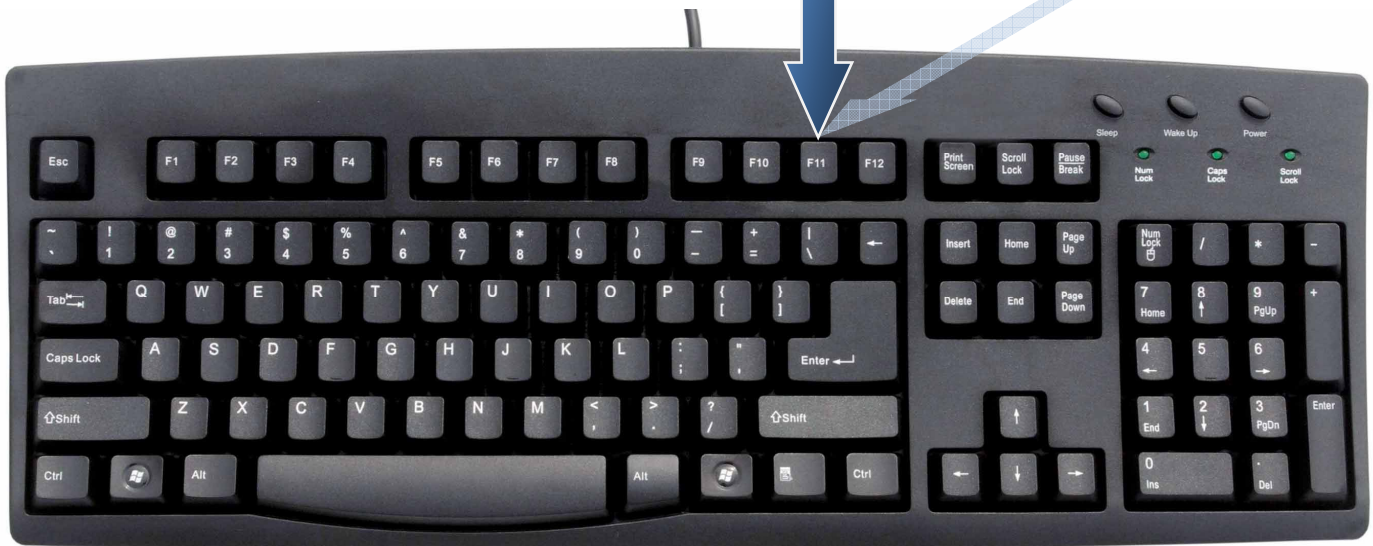

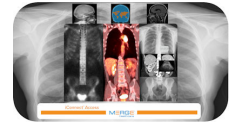



Open browser type: ipacs.chs-mi.com – Use Covenant User ID/PW

Search criteria screen Note: Study performed is a hover over field

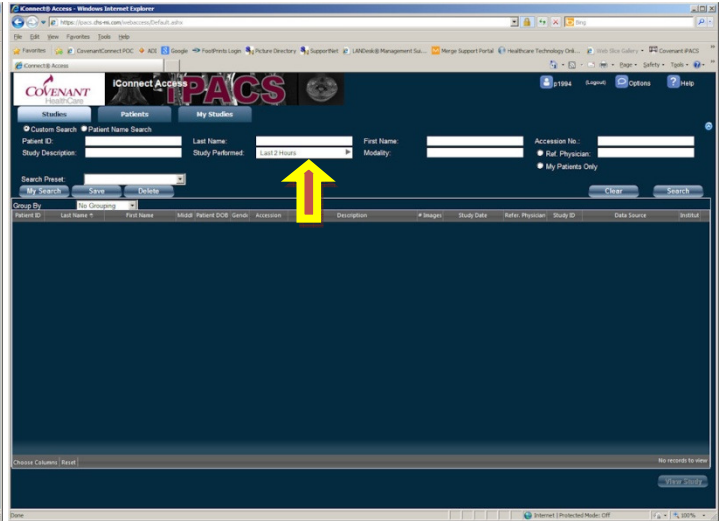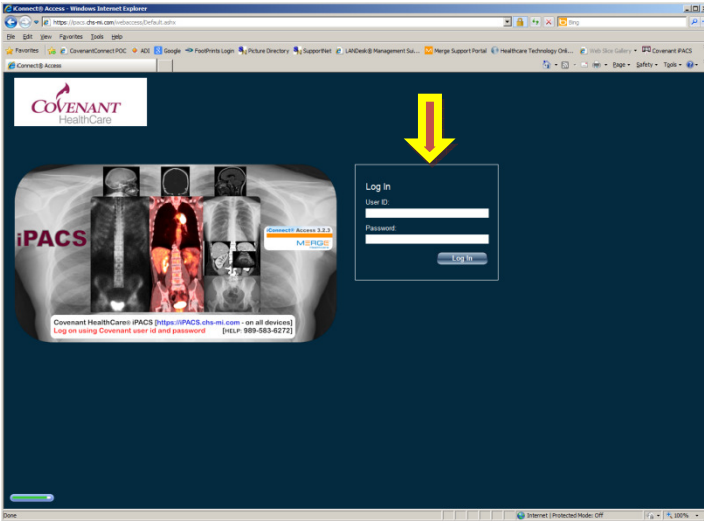



Set personal {User Preferences} under {Options}


{Group By} puts search results together in different ways

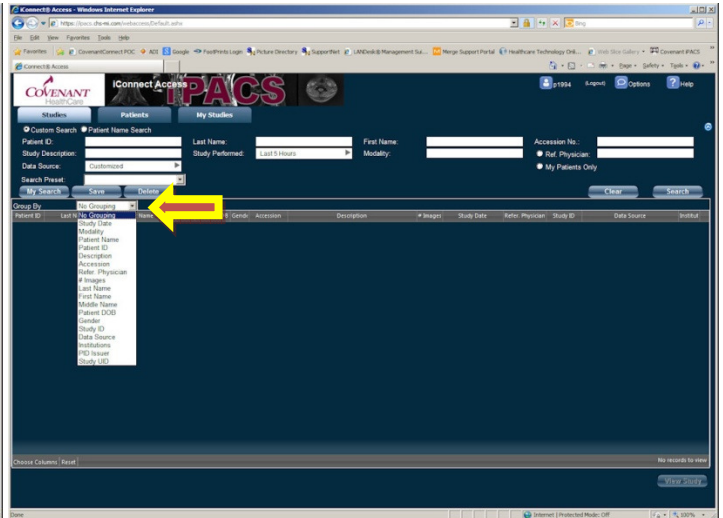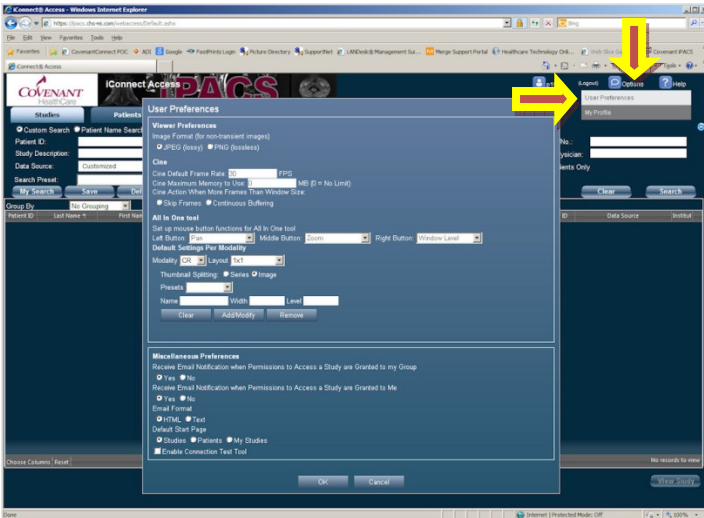


{Save} – input name – {Save} creates custom “Search Preset”

{Search Presets} are quickly selected through the drop down

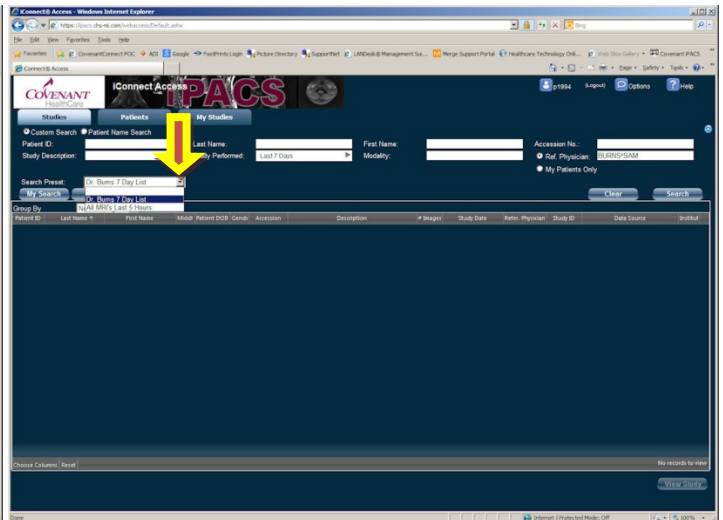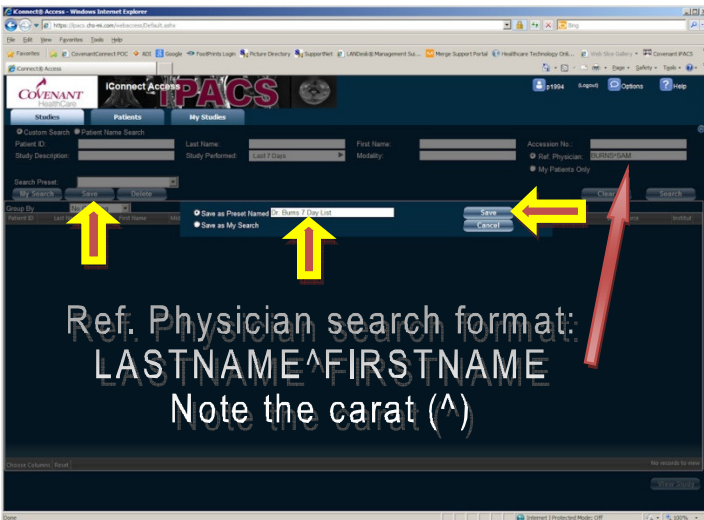


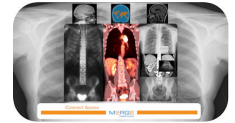



Input search criteria, click {Search}, double click study to open

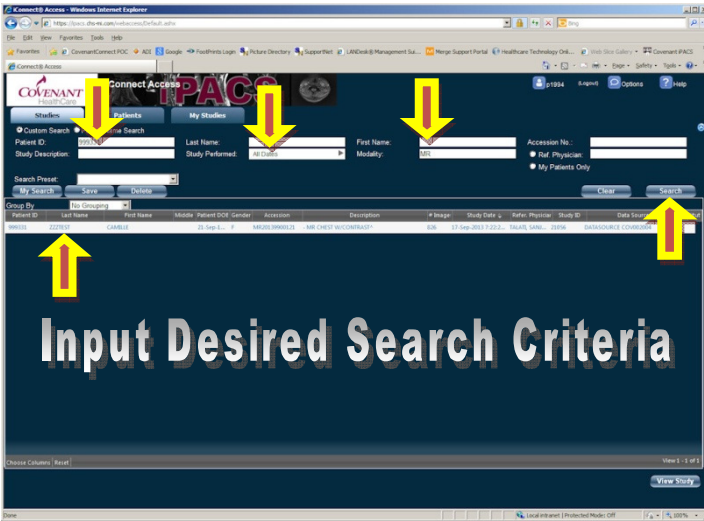


Drag & drop thumbnails into viewer - drag slider UP/DN to scroll

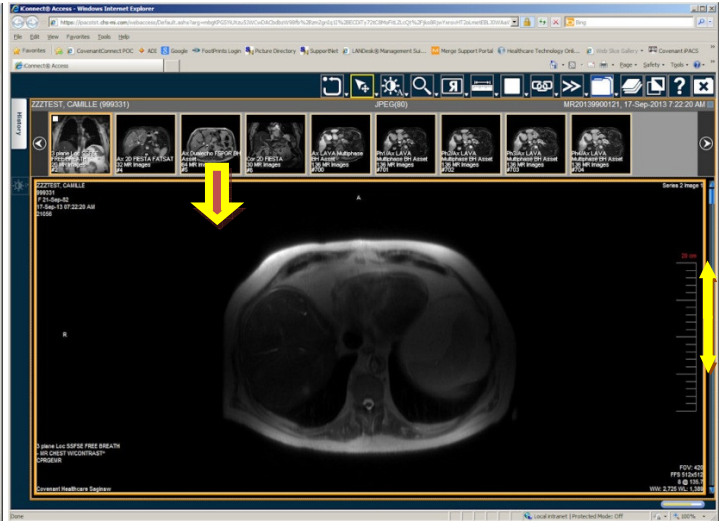


To set window/level use [Right] mouse button hold, move mouse -or- hover over W/L tools and choose {Auto Window Level}

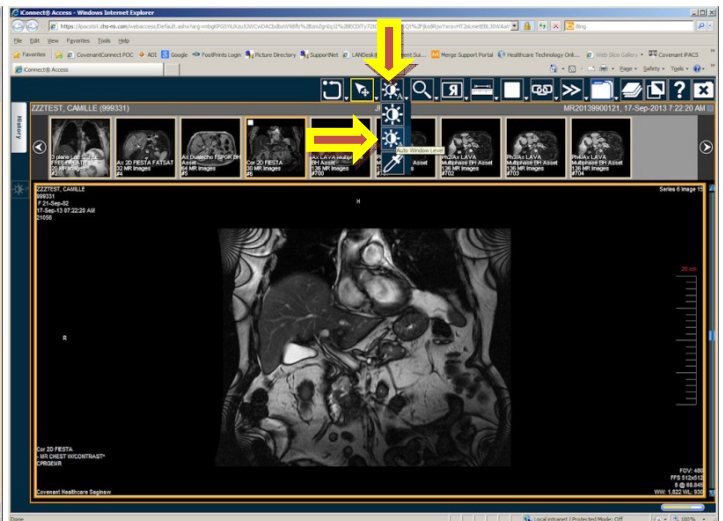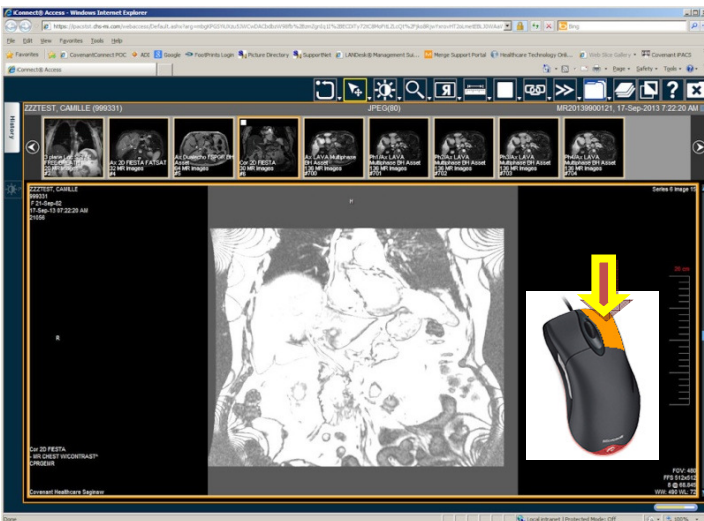


Click {History} tab to see study reports or other studies

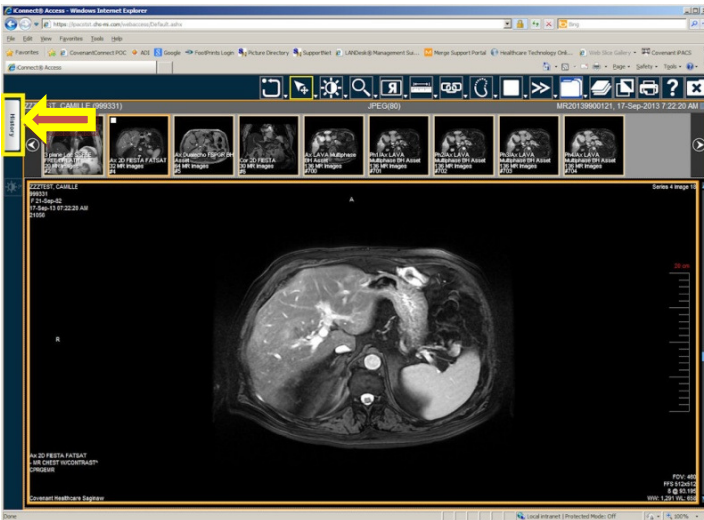



Click study to view reports - Double Click study to view images

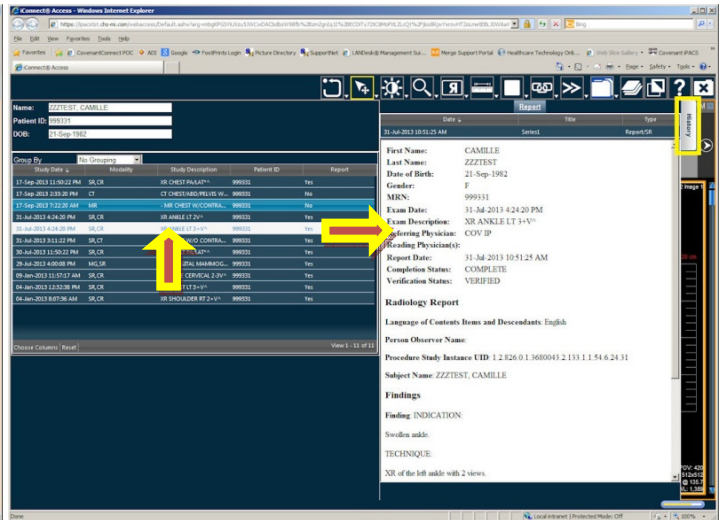

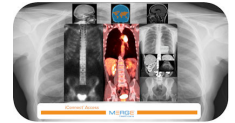



Viewed images open to the right of originally opened study

{Square Blue Icon} click toggles report open/close

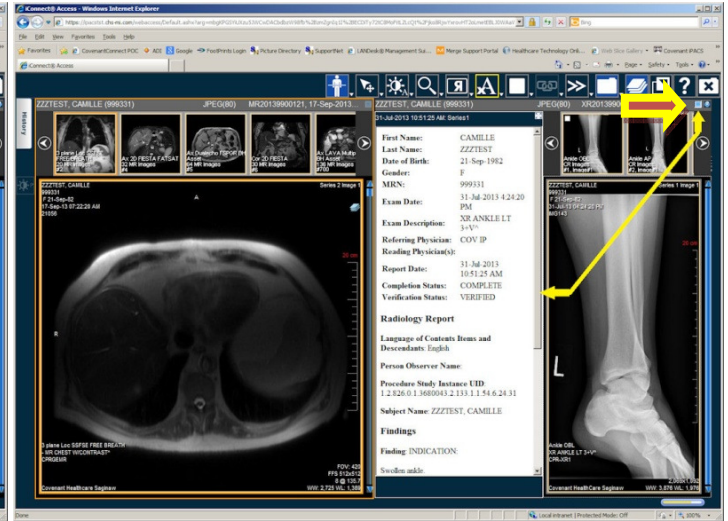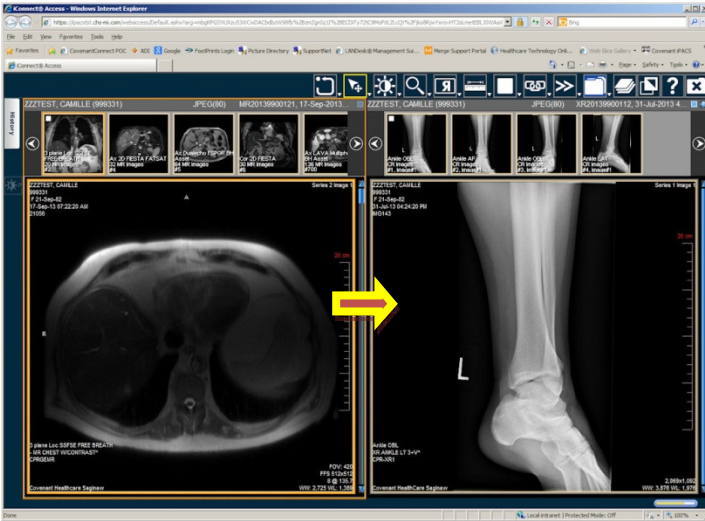



{Round Blue Icon} click closes alternate study viewer

Click {Magnify} tool then [Left] button hold to magnify/reduce

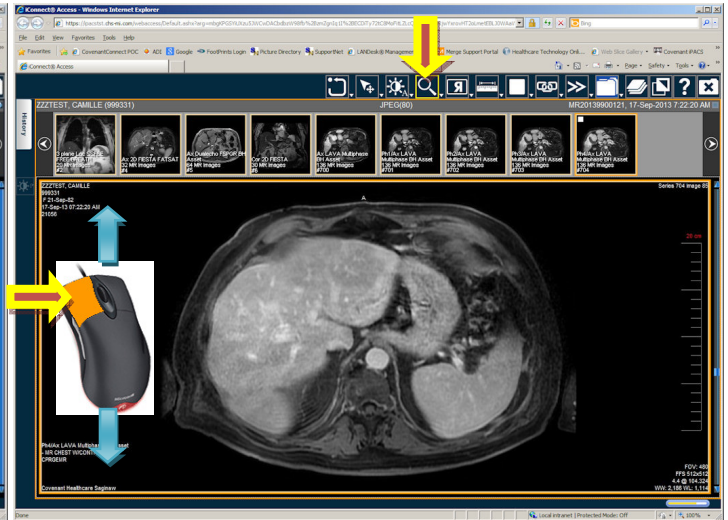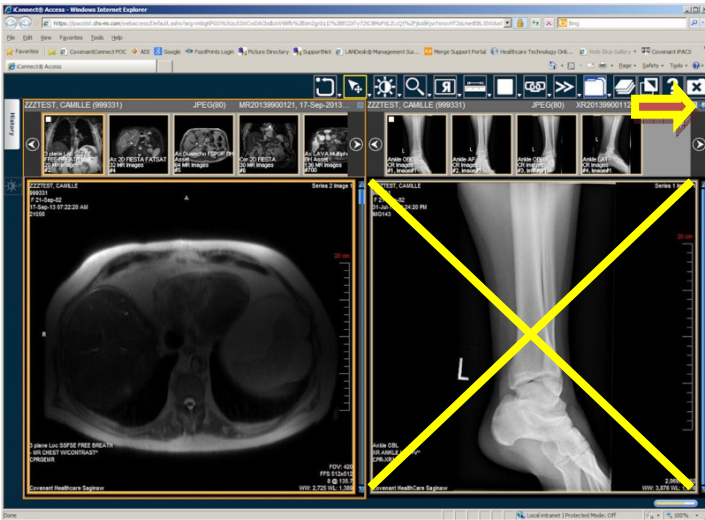

[F11] key press toggles full screen browser view on/off

{Full Screen} tool toggles full viewer on/off & buttons hide/show

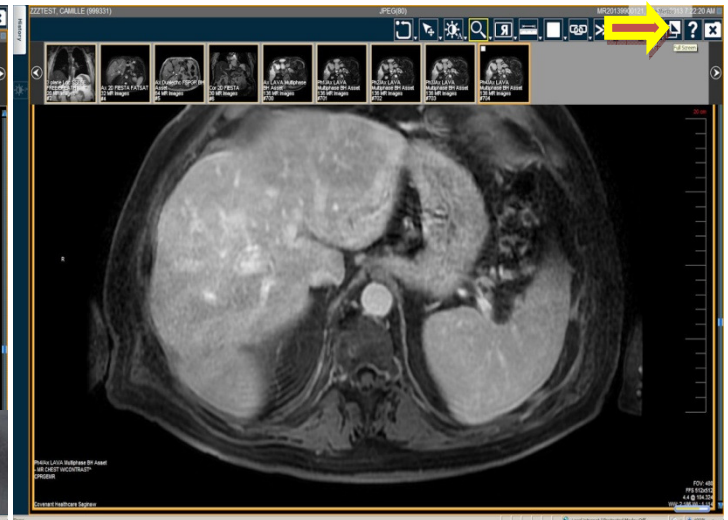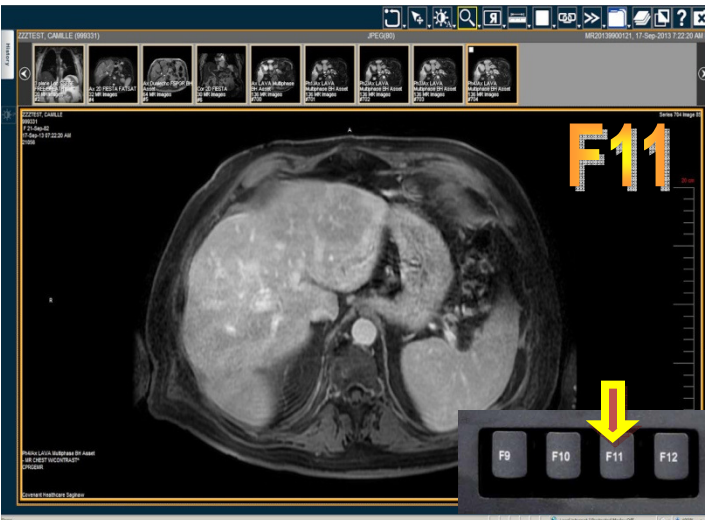




In full screen viewer {Show Menus} reveals tools/thumbnails




[F11] key press & {Full Screen} tool click reveals header/tools

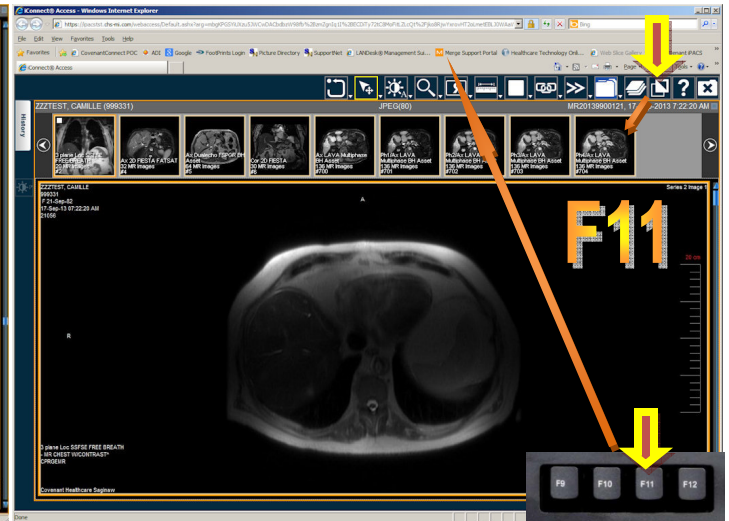


{Measurement} tool click followed by two place clicks measures




{Remove All Annotation} click clears measurements/annotations

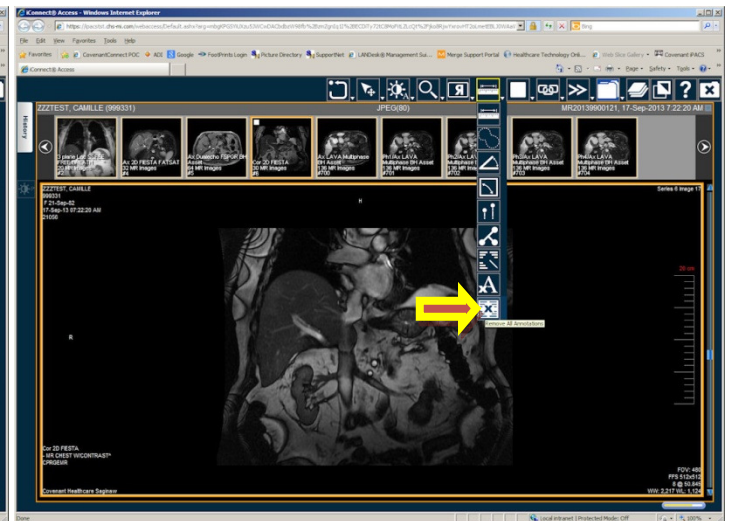


{Series Viewer} hover over then click splits the viewer frames

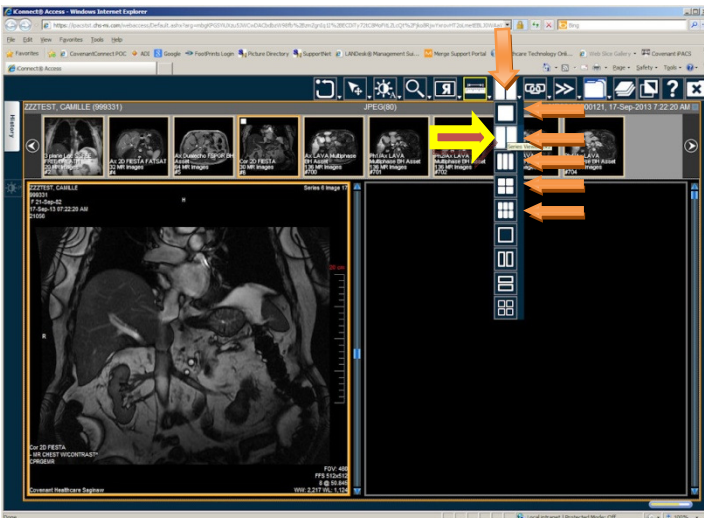


Active frame {Locator Line} tool click shows plane reference lines

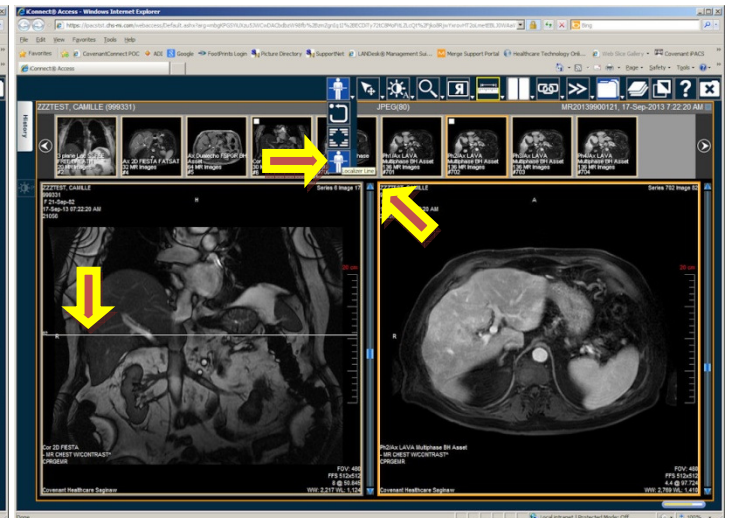

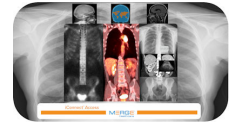


{Global Stack} click stacks all images in viewer for scroll through

{Toggle Text} click toggles annotations off/on

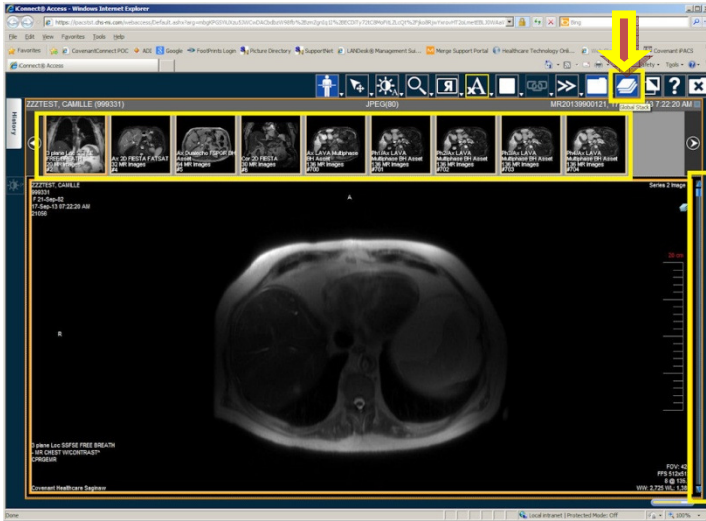


{Print} tool click prints or saves images <“Recommend text off”>

{Print} button click sends image to printer select window

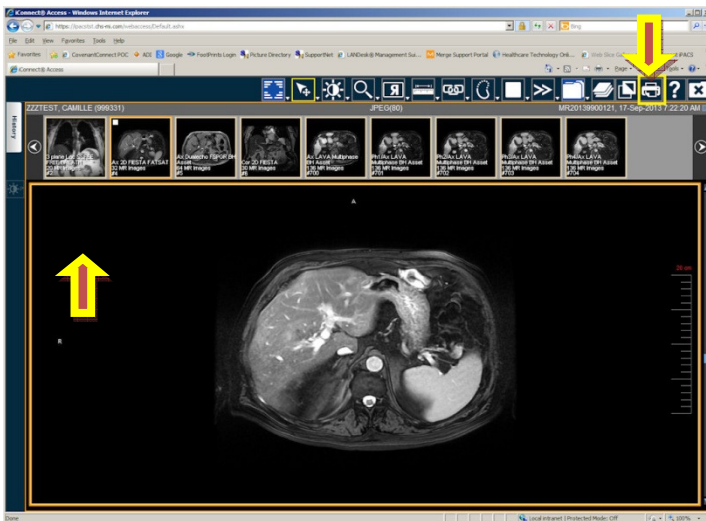


{Right} mouse button press to save image as a picture

{X} tool click closes study and returns to study query window

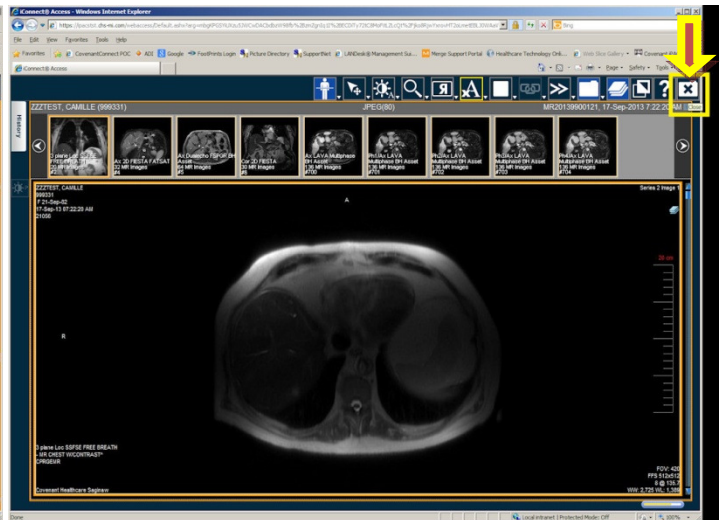



Click {Logout} to exit iPACS

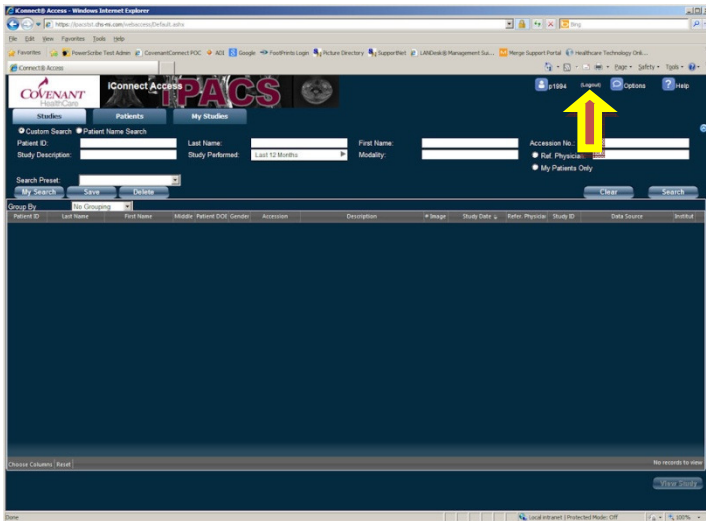

For assistance call:
Covenant I&D Support
989-583-6272