

Viewing Patient History Reports In iPACS – Sometimes A Two-Step Process

To view the report for a study being viewed click on the history tab. If there is a report associated with the viewed study it will show up immediately.

The screenshot displays the iPACS interface with a patient history table and a detailed report view. The table lists various studies, and the report view shows details for a specific study. Red arrows highlight the 'XR SHOULDER RT 2+V^' study description in the table and the corresponding 'XR SHOULDER RT 2+V^' exam description in the report. A red arrow also points to the 'History' tab on the right side of the interface.

Study Date	Modality	Study Description	Patient ID	Report
03-Nov-2013 1:04:48 AM	CR	XR SHOULDER RT 2+V^	317020	Yes
02-Jun-2012 10:18:23 PM	SR,PR,KO,CT	CT HEAD W/O CONTRAST^	317020	Yes
02-Jun-2012 9:06:31 PM	SR,CR	XR CHEST FRONTAL 1V^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR HIP RT 2+V^	317020	Yes
07-Feb-2012 8:41:36 PM	SR,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
23-Jan-2010 9:00:06 PM	SR,PR,KO,CT	CT CTA CHEST W/ W/O CONTR...	317020	Yes
23-Jan-2010 8:20:45 PM	US,SR,PR,KO	US RT UPPER QUADRANT^	317020	Yes

Report Details:

- Exam Date:** 03-Nov-2013 1:04:48 AM
- Exam Description:** **XR SHOULDER RT 2+V^**
- Referring Physician:** [Redacted]
- Reading Physician(s):** [Redacted]
- Report Date:** 03-Nov-2013 10:57:39 PM
- Completion Status:** COMPLETE
- Verification Status:** VERIFIED

Radiology Report

Language of Contents Items and Descendants: English

Person Observer Name:

Procedure Study Instance UID: 1.2.826.0.1.3680043.2.133.1.3.18.1.172.153474

Subject Name: [Redacted]

Findings

Finding: TECHNIQUE:

3 views of the right shoulder.

INDICATION:

[Redacted]

COMPARISON:

None.

Viewing Patient History Reports In iPACS – Sometimes A Two-Step Process

To view a report on another study for the same patient, a two step process is involved if there is more than one “Structured Report”. Step 1 is to click on the study itself. However, the report from the previous study will still be showing if there are multiple “Structured Reports”.

The screenshot displays the iPACS interface. On the left, a table lists patient studies. The table has columns for Study Date, Modality, Study Description, Patient ID, and Report. The study dated 02-Jun-2012 9:06:31 PM with the description 'XR CHEST FRONTAL LV^' is highlighted. A red arrow points from this row to the 'Exam Description' field in the report details on the right, which shows 'XR SHOULDER RT 2+V^'. Another red arrow points from the 'Date of Birth' field in the report details to the 'Date' column in the table, which shows '02-Jun-2012 9:50:20 PM'. A red box highlights the 'Date' and 'Type' columns in the table header. A red text box on the right contains the following text: 'Incorrect report with selected exam. MUST also click the report above as there are multiple structured reports available and it is expecting user to choose one.'

Study Date	Modality	Study Description	Patient ID	Report
03-Nov-2013 1:04:48 AM	CR	XR SHOULDER RT 2+V^	317020	Yes
02-Jun-2012 10:18:23 PM	SR,PR,KO,CT	CT HEAD W/O CONTRAST^	317020	Yes
02-Jun-2012 9:06:31 PM	SR,CR	XR CHEST FRONTAL LV^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR HIP RT 2+V^	317020	Yes
07-Feb-2012 8:41:36 PM	SR,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
23-Jan-2010 9:00:06 PM	SR,PR,KO,CT	CT CTA CHEST W/ W/O CONTR...	317020	Yes
23-Jan-2010 8:20:45 PM	US,SR,PR,KO	US RT UPPER QUADRANT^	317020	Yes

Report Details:

Exam Description: XR SHOULDER RT 2+V^

Exam Date: 03-Nov-2013 1:04:48 AM

Report Date: 03-Nov-2013 10:57:39 PM

Completion Status: COMPLETE

Verification Status: VERIFIED

Radiology Report

Language of Contents Items and Descendants: English

Person Observer Name:

Procedure Study Instance UID: 1.2.826.0.1.3680043.2.133.1.3.18.1.172.153474

Subject Name:

Findings

Finding: TECHNIQUE:

3 views of the right shoulder.

INDICATION:

COMPARISON:

None.

Viewing Patient History Reports In iPACS – Sometimes A Two-Step Process

Step 2 is to click on the actual report in the upper right report window. Once clicked the report for that study shows up in the patient history viewer window.

The screenshot displays the iPACS software interface. On the left, a table lists patient studies. The study 'XR CHEST FRONTAL 1V^' is highlighted. A red arrow points from this study to the 'Report' window on the right. In the 'Report' window, the report title 'XR CHEST FRONTAL 1V^' is highlighted, and another red arrow points to the 'Findings' section.

Study Date	Modality	Study Description	Patient ID	Report
03-Nov-2013 1:04:48 AM	CR	XR SHOULDER RT 2+V^	317020	Yes
02-Jun-2012 10:18:23 PM	SR,PR,KO,CT	CT HEAD W/O CONTRAST^	317020	Yes
02-Jun-2012 9:06:31 PM	SR,CR	XR CHEST FRONTAL 1V^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
16-Feb-2012 3:45:14 PM	SR,PR,KO,CR	XR HIP RT 2+V^	317020	Yes
07-Feb-2012 8:41:36 PM	SR,CR	XR ABD FLAT/UPRIGHT ^^	317020	Yes
23-Jan-2010 9:00:06 PM	SR,PR,KO,CT	CT CTA CHEST W/ W/O CONTR...	317020	Yes
23-Jan-2010 8:20:45 PM	US,SR,PR,KO	US RT UPPER QUADRANT^	317020	Yes

Report

Date: 02-Jun-2012 9:50:20 PM
Title: unknown
Type: Report/SR

Series1
Series1

First Name: [REDACTED]
Last Name: [REDACTED]
Date of Birth: [REDACTED]
Gender: F
MRN: [REDACTED]
Exam Date: 02-Jun-2012 9:06:31 PM
Exam Description: XR CHEST FRONTAL 1V^
Referring Physician(s): [REDACTED]
Reading Physician(s): [REDACTED]
Report Date: 02-Jun-2012 9:50:20 PM
Completion Status: COMPLETE
Verification Status: VERIFIED

Radiology Report

Language of Contents Items and Descendants: English

Person Observer Name: [REDACTED]

Procedure Study Instance UID: 1.2.826.0.1.3680043.2.133.1.3.18.1.150.1542

Subject Name: [REDACTED]

Findings

Finding: INDICATION:
A [REDACTED]

TECHNIQUE:
Portable AP one view chest.

FINDINGS:
The lungs do not show evidence of any confluent opacity. Cardiac and mediastinal silhouette is unremarkable. Bony structures are unremarkable.